# seca 956

<b>D</b>	<b><i>Bedienungsanleitung und Garantieerklärung</i></b>	<b>3</b>
<b>GB</b>	<b><i>Instruction manual and guarantee</i></b>	<b>13</b>
<b>F</b>	<b><i>Mode d'emploi et garantie</i></b>	<b>23</b>
<b>I</b>	<b><i>Manuale di istruzioni e garanzia</i></b>	<b>33</b>
<b>E</b>	<b><i>Manual de instrucciones y garantía</i></b>	<b>43</b>
<b>DK</b>	<b><i>Betjeningsvejledning og garantibevis</i></b>	<b>53</b>
<b>S</b>	<b><i>Bruksanvisning och garanti</i></b>	<b>63</b>
<b>N</b>	<b><i>Bruksanvisning og garantierklæring</i></b>	<b>73</b>
<b>FIN</b>	<b><i>Käyttöohje ja takuu</i></b>	<b>83</b>
<b>NL</b>	<b><i>Bedieningshandleiding en garantieverklaring</i></b>	<b>93</b>
<b>P</b>	<b><i>Instruções de utilização e declaração de garantia</i></b>	<b>103</b>
<b>GR</b>	<b><i>Οδηγίες χειρισμού και εγγύηση</i></b>	<b>113</b>
<b>PL</b>	<b><i>Instrukcja obsługi i gwarancja</i></b>	<b>123</b>

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---

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# Full certification

---

seca products you are purchasing not only technology matured over a century, but also quality confirmed by authorities, the legal system and by institutes.

seca products comply with European directives, standards and national laws. With seca, you are buying the future.

The products in these operating instructions comply with German legislation governing medical products and therefore with Directive 93/42/EEC of the Council of the European Community which is embodied in national laws throughout Europe.

Scales carrying this symbol comply with the strict requirements of Class III of the directive and can be used for medical measurements which have to be calibrated.

Device conforms to EU directives

- **M**: Conformity label according to Directive 2014/31/EU governing non-automatic weighing instruments (verified models)
- **18**: (Example: 2018) Year in which conformity verification was performed and the CE label was applied (verified model)
- **0102**: Notified body metrology (verified models)
- **0123**: Notified body medical products

seca's professionalism has also been officially recognized. The certificate from TÜV Product Service, the body responsible for medical devices, confirms that as a manufacturer of medical devices, seca rigorously complies with stringent legal requirements. seca's quality assurance system covers the design, development, production, sales and service of medical scales and measuring systems.

seca helps the environment.

We are concerned to save natural resources. We therefore strive to save packaging material where meaningful. What is left over can conveniently be disposed of locally via the German Dual System (recycling programme).

# 1. Congratulations!

---

By purchasing the **seca 956** electronic chair scale, you have acquired a highly-accurate and sturdy piece of equipment.

seca has been putting its experience at the service of health for over 150 years now, and as market leader in many countries of the world, is always setting new standards with its innovative developments for weighing and measuring.

In accordance with national regulations, the **seca 956** wheelchair scale with standing aid is used primarily in hospitals, doctors' surgeries and in-patient nursing facilities. The scale is a Class I medical product, and has been calibrated in accordance with the precision class III.

Weight is determined within a few seconds.

The **seca 956** can be moved on casters and its low power consumption means it will provide mobile service for a long period.

The **seca 956** chair scale is made for people who are not especially steady on their feet. To give you a safe feeling while weighing, the chair scale has a solid tubular frame and a firmly bolted seat. The free-running casters make the scale very easy to move. The wheel-locking lever, together with the swiveling armrests and foot supports, makes it easier to sit in safety.

The scale is of a very sturdy construction and will give you long and loyal service. It is easy to operate and the large display is easy to read.

## 2. Safety

---

Before using the new scale, please take a little time to read the following safety instructions.

### Warnings

- Never use the **seca 956** chair scale for transport purposes. Do not use **seca 956** chair scale to transport people or objects. If you transport people or objects using the chair scale, there is an acute risk of injury.
- Never place the **seca 956** chair scale on sloping surfaces. The brakes on the casters could be released.
- Prevent the scale from rolling away accidentally. Use the wheel brake when the patient is sitting down or getting up and assist people who are unsteady on their feet.

### Notes

- Follow the safety instructions in the instruction manual.
- Keep the operating manual and the declaration of conformity contained therein in a safe place.
- Have scale serviced and re-calibrated on a regular basis (see "Servicing/verification" on page 21).
- Use only the type of battery stated (see "Unpacking" on page 15).

# 3. Before you start...

---

## Unpacking

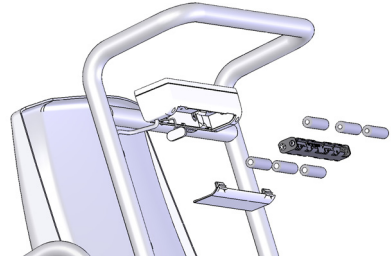
The scope of supply includes:

- Chair scale comes complete assembled
- Battery holder containing 6 mignon-type AA 1.5 V batteries
- 1 set of operating instructions

## Power supply

The scale is powered exclusively by batteries.

- Open the battery compartment in the base of the display housing.
- Connect the cable to the holder and batteries.
- Place the holder in the battery compartment.
- Close the lid of the battery compartment again.

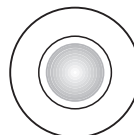
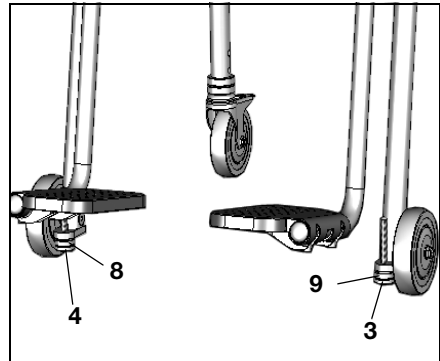
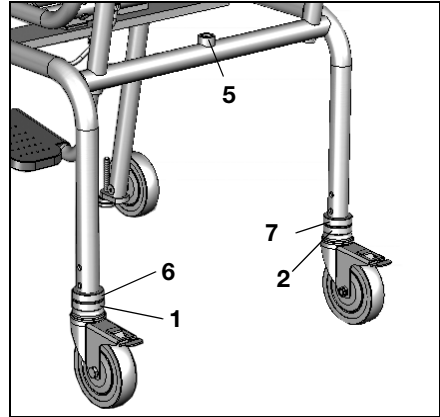
The LC display has a low power consumption – approx. 4.000 weighing operations can be performed with one set of batteries.

Once battery voltage is no longer adequate, the battery symbol  first appears in the display to indicate that battery power is dropping.

Once battery power is no longer sufficient, **bAtE** appears in the display and weighing is no longer possible.

## 4. Setting up and aligning the scale

- Place the scale on a level surface.
- Align the scale using the two bottom knurled nuts (1 and 2) and the two adjustable feet (3 and 4) so that the display in the spirit level (5) is central. This setting can be secured using the top lock-nuts (6, 7, 8 and 9).

spirit level

# 5. Operation

## Controls and displays

Controls on the display

Switch scale on and off

set display to zero, activate/deactivate tare function

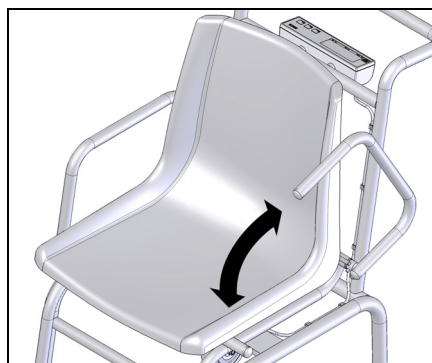
activate hold function

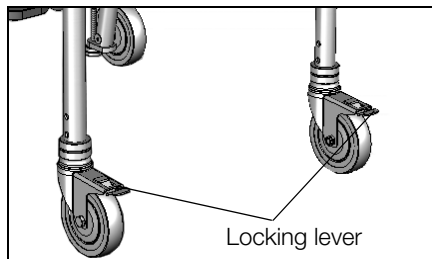
## Other controls

The armrests on the chair scale can be swivelled backwards independently. This is useful because it makes it easier for the person being weighed to sit down.

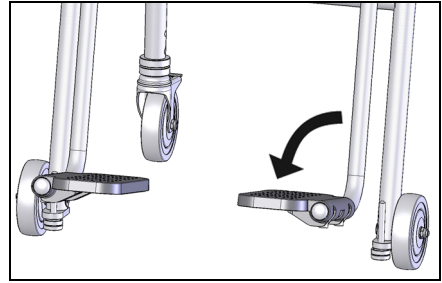
- Ensure that you do not trap any part of the body or pieces of clothing when folding the rests down.

- The rear casters can be locked if you push down the two locking levers on the casters.

- Once the person who is to be weighed is seated, the foot-rests are pulled out and tilted downwards. The patient can now place his or her feet on the foot-rests.

## Correct weighing

- Press the **start** key with no load on the scale.  
**SECA**, **88888** and **00** appear consecutively in the display. The scale is then automatically set to zero and ready to use.
- Get the patient to sit on the scale.
- The patient's weight can only be determined once he or she is no longer in contact with the floor. Be sure, therefore, that the patient's feet remain on the footrests throughout the weighing operation.
- Read the weighing result off the digital display.
- If a load greater than 200 kg is placed on the scale, **StoP** appears in the display.  
 Take the weight off the scale when the word **StoP** appears in the LC display.
- To switch off the scale, press the **start** key again.

**StoP**

## Automatic switch-off/economy mode

The scale switches off automatically after approx. 3 minutes if it has no load on it. If the TARE or HOLD functions are activated, or at the minimum load of 10 kg, the scale switches off after approx. 2 minutes.

## Taring weight display (tare)

The Tare function is very useful if an additional weight (a blanket, for example) is to be ignored for weighing purposes.

Proceed as follows:

- Press the **start** key with no load on the scale.

- First place the additional weight (a blanket, for example) on the scale and activate the Tare function by pressing the **tare** key.

The display is reset to zero. **0.0** appears in the display. The note "NET" appears.

- Now put the patient and the additional weight on the scale.

The scales calculate the patient's weight without the additional weight. You can now perform as many weighing operations as you like.

- Press the **tare** key again to deactivate the Tare function; the scale is then back in normal weighing mode and the tare value has been erased.
- The tare value is likewise erased by the scale being switched off.

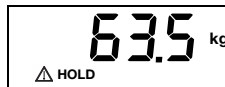
## Store weight value (hold)

You can also continue displaying the weight determined once the load has been removed. This makes it possible to look after a patient first, before having to note the weight:

- Press the **hold** key whilst the load is on the scale.

"HOLD" appears in the digital display. The value is stored until the scale switches off automatically.

### Note:

The HOLD function cannot be calibrated. This is indicated by the  $\Delta$  symbol in the display.

- The HOLD function can be switched off by pressing the **hold** key again.

**0.0** appears in the display if the scale has no load on it, otherwise the current weight. The "HOLD" display vanishes.

### Note:

**The HOLD function can also be used if the TARE function has been executed beforehand.**

## 6. Cleaning

---

Clean the seat and the housing as required using a domestic cleaning agent or commercially-available disinfectant. Follow the manufacturer's instructions.

Under no circumstances use abrasive or acid cleaners, white spirit, benzene or the like for cleaning. Such substances can damage the high-quality surfaces.

## 7. What do I do if...

---

### ... no weight display comes on?

- Is the scale switched on?
- Check the batteries.

### ... 0.0 does not appear before weighing?

- Switch off the scale with the Start key and start the scale again – there must not be any load on the scale – and only its feet should be in contact with the floor.

### ... one segment is illuminated constantly or not at all?

- The relevant segment has a fault. Inform Servicing.

### ... the display shows ?

- Battery voltage is running low. Change the batteries in the next few days.

### ... bAtt appears in the display?

- Batteries are empty. Put in new batteries.

### ... 5tOP appears in the display?

- Maximum load has been exceeded. Take the weight off the scale.

### ... the display flashes?

- If you have just activated the Hold or Tare function, please wait until the scale has detected a steady weight value. The display will then stop flashing.
- If there has been no activation of a function beforehand, remove the load from the scale and wait until 0.0 is displayed, then weigh again.

### ... the display Er and a number appear (e.g. Er: 12)?

- Switch off the scale with the Start key and start the scale again. The scale will then work normally again.

If this is not the case, disconnect the power supply by briefly removing the batteries. If this measure is equally unsuccessful, inform the Service department.

## 8. Servicing/verification

---

Have the scale verified by authorised persons in accordance with national legal requirements. Verification is essential if one or more security marks are damaged.

We recommend having your scale serviced by your local service agent before having it verified. seca Customer Service will be pleased to give you further assistance.

## 9. Technical data

---

Maximum load	200 kg
Minimum load	2 kg
<b>Graduations</b>	100 g
<b>Dimensions</b>	
height:	906 mm +35,7 mm
width:	563 mm
depth:	946 mm
Seat height:	520 mm
<b>Height of figures</b>	20 mm
Weight	approx. 22.4 kg
Temperature range	+10 °C to +40 °C
Power supply	batteries
<b>Batteries</b>	6 type AA 1.5 V batteries
<b>Power consumption</b>	approx. 25 mA
<b>Calibration as per Directive 2014/31/EU:</b>	calibrated, Class III
<b>Medical product as per Directive 93/42/EEC</b>	Class I
<b>EN 60 601-1:</b>	
Type B electromedical device:	
<b>Accuracy at first calibration</b>	Range 2-50 kg: ±50 g Range 50-200 kg: ±100 g
<b>Functions</b>	TARE, HOLD

# 10. Disposal

---

## Disposing of the device

Do not dispose of the device in domestic waste. The device must be disposed of properly as electronic waste. Follow the national regulations which apply in your case. For further information, contact our service department at: [service@seca.com](mailto:service@seca.com)

## Batteries

Do not throw used batteries away in domestic waste. Dispose of batteries at collection points in the vicinity. When buying new batteries, select those low in harmful substances and containing no mercury (Hg), cadmium (Cd) or lead (Pb).

# 11. Warranty

---

A one-year warranty from date of delivery applies to defects attributable to poor materials or workmanship. All moveable parts - batteries, cables, mains units, rechargeable batteries etc. - are excluded. Defects which come under warranty will be made good for the customer at no charge on production of the receipt. No further claims can be entertained. The costs of transport in both directions will be borne by the customer should the equipment be located anywhere other than the customer's premises. In the event of transport damage, claims under warranty can be

honoured only if the complete original packaging was used for any transport and the product secured and attached in that packaging just as it was when originally packed. All the packaging should therefore be retained.

A claim under warranty will not be honoured if the equipment is opened by persons not expressly authorised by seca to do so.

We would ask our customers abroad to contact their local sales agent in the event of a warranty matter.

# 12. Declaration of conformity

---

seca gmbh & co. kg hereby declares that the product meets the terms of the applicable European directives. The unabridged declaration of conformity can be found at: [www.seca.com](http://www.seca.com).

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