Sizes:
The 1-part MECRON knee braces need to be ordered in relation to upper and lower leg circumference. Measure the circumference both 20 cm above and below the middle of the patella.

The 3-part MECRON knee braces are individually adjustable to the patient's leg circumference (up to a max. thigh circumference of 74 cm).

1-part knee braces:

MECRON Knee Flexion & MECRON Knee Straight

Length	Size	Girth Femur	Girth Lower Leg	Colours
40 cm	S	43 cm	30 cm	Jeans Blue,
	M	48 cm	35 cm	Vinyl Blue,
	L	53 cm	40 cm	Black
50 cm	S	44 cm	23 cm	
	M	49 cm	28 cm	
	L	54 cm	33 cm	
	XL	59 cm	38 cm	
60 cm	S	45 cm	22 cm	
	M	50 cm	27 cm	
	L	55 cm	32 cm	
	XL	60 cm	35 cm	

3-part knee braces:

MECRON Knee 3-part

Length	Beugung	Colours
40 cm	0°/20°	Velour Blue,
50 cm	0°/20°	Black
60 cm	0°/20°	

MECRON Knee Clinical

Length	Beugung	Colours
50 cm	0°/20°	Black-Yellow
60 cm	0°/20°	

MECRON Knee Pediatric

Length	Colours
22 cm	Blue-Yellow
32 cm	Blue-Orange,
42 cm	Blue-Light Blue

Material:
Outer material: vinyl, denim or velour
Inner material: PU soft foam
Textile covering: synthetic yarns
Edging strips: 100% cotton
Klettverschlüsse: Polyamide
Sewing threads: polyester

Care: The product is maintenance-free.

Caution

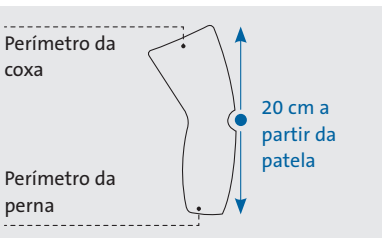
Always follow these instructions for use. If you notice any side effects (e.g. swollen skin, pressure points, disrupted blood circulation, etc.) please consult your doctor. If used improperly, the product's function and optimum protection can no longer be guaranteed. This appliance is intended to assist you in your recovery and should only be used as prescribed by (or in consultation with) your doctor or medical specialist. DARCO (Europe) GmbH does not assume any responsibility.

Avoid using the brace in water and do not expose it to acids and solvents. It is advisable to wear dressings. We recommend that you refrain from driving while you are wearing this appliance. This appliance is a single-use product.

Cleaning instructions

All MECRON knee braces are washable at 30°. Make sure not to spin the braces. Remove the aluminium rods and close the velcro-fasteners prior to washing. Wash braces made of denim separately as the dye in this material may run when it is washed. After washing, pull the braces into shape, re-insert the rods and place horizontally to dry.

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MECRON by DARCO

Indications

- > Conservative therapy in case of collateral ligament injuries
- > Cruciate ligament instabilities
- > Meniscus injuries
- > Traumatic or degenerative changes to knee joints
- > Immobilisation nach Patella dislocation
- > Posttraumatic, postoperative immobilisation
- > Immobilisation after ligament injuries
- > Conditions after meniscus suture
- > Preoperative pain

- Contraindications:**
- > No contraindications are known.

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