

# HARTMANN



# Bacillo<sup>®</sup> 30 Sensitive

## Powerful yet gentle surface disinfection



## New highlights:



### Optimum effectiveness

- effective against coronaviruses
- virucidal activity against enveloped viruses acc. EN 16777
- effective against noroviruses in 2 minutes
- tuberculocidal and mycobactericidal efficacy in 3 minutes



### Gentle on materials

- for displays, keyboards, mobile phones, tablets
- sensitive, non-invasive medical product



### Plenty of coverage

- study shows: Bacillo<sup>®</sup> 30 Sensitive Tissues disinfects up to 30 m<sup>2</sup> more than comparable products [1]



### Best cleaning performance

- study shows: Bacillo<sup>®</sup> 30 Sensitive Tissues have best cleaning performance compared to the competition [2,3]



### Gentle on skin

- certified by SGS\* Institut Fresenius [4]
- ECARF-certified [5,6]: the 1st professional allergy- and asthma-friendly surface disinfection
- user-friendly, as it is free from dyes and perfumes, and can be used without gloves\*\*

\* the world's leading testing, inspection and certification company  
\*\* if infection control and occupational safety protocols permits it



### Future-oriented

- responsible and conservative use of active ingredient with 60% less surfactants/residue\*\*\*

\*\*\* Compared to Bacillo<sup>®</sup> 30

# Bacillo® 30 Sensitive – rapid disinfection for even sensitive surfaces and extraordinarily gentle on materials, highly effective and particularly user-friendly

| Proven efficacy   |        |
|---|--------|
| Virucidal against enveloped viruses (EN 14476)          | 1 min  |
| Virucidal against enveloped viruses (EN 16777)          | 1 min  |
| Bactericidal + Yeasticidal (EN 13727/EN 13624/EN 16615) | 1 min  |
| Norovirus (EN 14476)                                    | 2 min  |
| Tuberculocidal activity (EN 14348)                      | 3 min  |
| Mycobactericidal activity (EN 14348)                    | 3 min  |
| Limited spectrum virucidal activity (EN 14476)          | 30 min |
| Adenovirus (EN 14476)                                   | 30 min |
| Rotavirus (EN 14476)                                    | 1 min  |



## The Bacillo® 30 Sensitive range

|   | Package content           | Item number | Packaging unit |
|---|---------------------------|-------------|----------------|
| <b>Bacillo® 30 Sensitive Tissues</b>        | Flow pack with 80 Tissues | XXX XXX     | 6              |
|   | Flow pack with 24 Tissues | XXX XXX     | 12             |
| <b>Bacillo® 30 Sensitive Tissues in XXL</b> | Flow pack with 40 Tissues | XXX XXX     | 6              |
| <b>Flow pack wall holder</b>                | 1 Wall holder             | XXX XXX     | 1              |
| <b>Bacillo® 30 Sensitive Foam</b>           | 750 ml bottle             | XXX XXX     | 8              |
|   | 5 litre canister          | XXX XXX     | 1              |



Use surface disinfectants safely. Always read the label and product information before use.

- 1 Flowpack with 80 Tissues. DITF; Service Center Test Technologies Schweins M, E-0130-TT-20 (2020). Unpublished study – data available at BODE Chemie GmbH.
- 2 In the removal of a fat-dust soil, test report WL 2566A/21 of 29.04.2021, „Comparative investigation of the cleaning performance of 14 ready-to-use wet disinfectant wipes“, wfk – Institut für Angewandte Forschung GmbH, Krefeld. Unpublished study – data available at BODE Chemie GmbH.
- 3 The tested ready-to-use wet disinfection wipes represent more than 90 % of the value share (as of Q1, 2021) of the German market in the segment of pre-soaked disinfection wipes.
- 4 Segger D. (2021): SGS Study No. 204-01-0029. Unpublished study – data available at BODE Chemie GmbH.
- 5 Zuberbier T (2021): ECARF Studie 033-R-21. Unpublished study – data available at BODE Chemie GmbH.
- 6 Zuberbier T (2021): ECARF Studie 021-S-21. Unpublished study – data available at BODE Chemie GmbH.



Helps. Cares. Protects.

PAUL HARTMANN AG  
Paul-Hartmann-Straße 12  
89522 Heidenheim  
Germany

Phone: +49-7321-36-0  
Fax: +49-7321-36-3636  
info@hartmann.info  
www.hartmann.info