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Instructions for Use EN

MTA cement, extra radiopacity, extra fast

Harvard MTA XR Fast OptiCaps® is a biocompatible repair cement in OptiCaps®. Due to its firm consistency in combination with a short setting time MTA XR Fast is the ideal choice for **root-end filling (retrograde)**.

Harvard MTA XR Fast powder consists of very fine hydrophilic particles of several hydraulic mineral oxides. After contact with the liquid it forms a gel that hardens to an impermeable barrier.

Harvard MTA XR Fast OptiCaps® are easily activated and the content of the capsule is easily ejected with the Harvard Applier OptiCaps®. Mixing of Harvard MTA XR Fast OptiCaps® (mixing time 30 seconds) is achieved by a high frequency mixer with about 4,300 oscillations/minute.

Notes

- In the first hour after application handle the placed MTA cement carefully.
- Store Harvard MTA XR Fast OptiCaps® in the sealed packaging in a dry place prior to use.
- Harvard MTA XR Fast can cause discoloration.

Indications

- Repair of root perforations
- Root-end filling (retrograde)
- Pulp capping (direct and indirect)
- Apexification / Root-end filling (orthograde)
- Repair of internal resorption
- Pulpotomy

Contraindications / side-effects

None known.

1. Activation and mixing

See "Instructions for the activation and mixing of Harvard OptiCaps®".

Mixing time for the Harvard MTA XR Fast OptiCaps® is 30 seconds.

Attention:

Avoid lag times between the processes of activation, mixing and application as the material is in the process of setting and lag times may impair or prevent application of the material.

The material must be extruded within **10 seconds** after the end of mixing.

To prevent dehydration during setting, apply Harvard MTA XR Fast immediately after mixing.

2. Working time

Working time at 23 °C (from the start of mixing): **about 2 minutes**

3. Applications

3.1. Repair of root perforations

Place rubber dam and clean the root canal system using intracanal instruments and irrigate with NaOCl. Dry the root canal with paper points and isolate the perforation.

Fill the root canal apical of the perforation completely with a suitable root canal filling material.

Mix a capsule of Harvard MTA XR Fast OptiCaps® as described under item 1 and extrude it on a glass plate.

Apply Harvard MTA XR Fast with suitable instruments into the perforation site and condense it.

Check the position of Harvard MTA XR Fast in the root canal by an X-ray. If an adequate barrier has not been created, rinse the MTA cement out of the canal and repeat the procedure.

Remove excess moisture with a cotton pellet or a paper point.

Place a moist cotton pellet in the access to the root canal and apply a temporary filling material.

Alternatively seal the root canal with a suitable root canal filling material and seal the cavity with a tight filling.

Both options can be done at the earliest **3 minutes** after placement of the Harvard MTA XR Fast.

Harvard MTA XR Fast repair material remains as a permanent part of the root canal filling.

3.2. Root-end filling (retrograde)

Create a surgical access to the root-end and resect the root. Prepare an apical cavity to a depth of 3-5 mm.

Isolate the area and dry the root-end cavity with paper points.

Achieve hemostasis with suitable methods.

Mix a capsule of Harvard MTA XR Fast OptiCaps® as described under item 1 and extrude it on a glass plate.

Apply Harvard MTA XR Fast with suitable instruments and condense it using a small plugger.

Remove excess cement and clean the surface of the root with a moist piece of gauze.

Confirm placement of Harvard MTA XR Fast repair material with an X-ray. Harvard MTA XR Fast remains as a permanent part of the root canal filling.

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