

# schülke →



Ready to use disinfectant wipes  
impregnated with an alcoholic solution.

## mikrozid® AF wipes

**PRAXISDIENST**  
Medical Supplies since 1953  
**Order here!**

### Our Plus

- ready to use, impregnated disinfection wipes
- very broad biocidal activity within the shortest possible time
- dries rapidly
- wide range of wipe sizes and packaging

### Application areas

Alcohol-based rapid disinfection of medical devices in all areas and all kinds of wipeable surfaces with an increased risk of infection and where short contact times are required, e.g.:

- Patient nearby surfaces
- Patient treatment units
- Examination couches
- Operating tables and nearby work surfaces
- Surfaces of medical equipment

### Instructions for use

Thoroughly wipe the surface with the impregnated wipe and allow to take effect. Make sure to wet surfaces completely and keep them wet for the whole exposure time. Make sure that all visible dirt is removed prior to disinfection. When opening the lid and inserting the first cloth, make sure that hygiene requirements are maintained. Make sure to close the lid tightly after use. Do not treat surfaces that are sensitive to alcohol, e.g. acrylic glass. Not suitable for the disinfection of invasive medical devices. schülke guarantees, if used in the correct way, a shelf life of 3 months after the package has been opened.

### Microbiological efficacy

| Efficacy  | Concentration | Contact time |
|---|---------------|--------------|
| <b>bactericidal</b><br><i>in accordance with VAH Guidelines with short contact times</i>  | ready-to-use  | 1 min.       |
| <b>MRSA</b>   | ready-to-use  | 1 min.       |
| <b>mycobactericidal</b><br><i>in accordance with VAH Guidelines with short contact times - dirty conditions</i>   | ready-to-use  | 1 min.       |
| <b>fungicidal</b>   | ready-to-use  | 2 min.       |
| <b>levurocidal</b><br><i>in accordance with VAH Guidelines with short contact times</i>   | ready-to-use  | 1 min.       |
| <b>enveloped viruses</b><br>(incl. HIV, HBV, HCV)<br><i>in accordance with DVV (German Registered Association for Combating Viral Diseases)/RKI Guideline</i> | ready-to-use  | 30 sec.      |
| <b>Norovirus</b><br>EN14476   | ready-to-use  | 1 min.       |
| <b>Rotavirus</b>  | ready-to-use  | 30 sec.      |
| <b>Polyoma SV40</b><br><i>clean conditions</i>  | ready-to-use  | 10 min.      |
| <b>Adenovirus</b><br>(type 5)<br><i>in accordance with DVV (German Registered Association for Combating Viral Diseases)/RKI Guideline</i>                     | ready-to-use  | 5 min.       |

### Certificates

- VAH certificate



CE 0297



# mikrozid® AF wipes

## Product data

### Composition:

100g agent solution contain the following active ingredients:  
25 g Ethanol (94 %), 35 g Propan-1-ol

### Chemical-physical data

|                    |                                    |
|--------------------|------------------------------------|
| Color              | colourless                         |
| Density            | ca. 0,89 g/cm <sup>3</sup> / 20 °C |
| Flash point        | 27 °C / Method: DIN 51755 Part 1   |
| Form               | liquid                             |
| pH                 | Not applicable                     |
| Viscosity, dynamic | not determined                     |

## Special advice

### Use disinfectants safely. Always read the label and product information before use.

The product is highly compatible with metals and synthetics (except acrylic glass and alcohol-sensitive varnishes). Please follow carefully any national fire and explosion protection guidelines when using alcohol-based disinfectants. Only treat small surfaces. The wiping range depends on the room temperature and on the structure of the surface to be disinfected. More information available on request.

### Refilling of the tin

First disinfect hands and wear new protective gloves. During the refilling process, please ensure that the gloves are not contaminated by other activities. On every refill, reprocess the lid closure. In reprocessing, subject the lid closure to disinfectant cleaning with an alcohol based surface disinfectant.

Following sizes are available:

- mikrozid® AF wipes (10 x 150 wipes 14 x 18 cm) Surface coverage\*: approx. 0.3 m<sup>2</sup>
- mikrozid® AF wipes Refill (20 x 150 wipes 14 x 18 cm) Surface coverage\*: approx. 0.3 m<sup>2</sup>
- mikrozid® AF wipes *Jumbo* (10 x 200 wipes 20 x 27 cm) Surface coverage\*: approx. 0.5 m<sup>2</sup>
- mikrozid® AF wipes *Jumbo* Refill (12 x 200 wipes 20 x 27 cm) Surface coverage\*: approx. 0.5 m<sup>2</sup>
- mikrozid® AF wipes *premium* (12 x 50 wipes 20 x 20 cm) Surface coverage\*: approx. 0.5 m<sup>2</sup>

## Information for order

| Item   | Delivery form | Item no.   |
|--|---------------|------------|
| mikrozid® AF wipes premium softpack (50 wipes)   | 12/Cartron    | on request |
| mikrozid® AF wipes Jumbo tin (10 x 200 wipes)    | 10/Cartron    | on request |
| mikrozid® AF wipes tin (10 x 150 wipes)          | 10/Cartron    | on request |
| mikrozid® AF wipes Jumbo refill (20 x 200 wipes) | 1/Cartron     | on request |
| mikrozid® AF wipes refill (20 x 150 wipes)       | 1/Cartron     | on request |

These products are not available in every country. For more information please contact our local subsidiary or distributor.

## Accessories

| Accessories                             | Item no. |
|---|----------|
| wall bracket Jumbo tub                  | 134421   |
| Wall mounting for the dispenser (small) | 134417   |

## Related Products

- mikrozid® PAA wipes
- mikrozid® sensitive wipes
- mikrozid® universal wipes

## Environmental information

schülke manufactures products economically and with advanced, safe and environmentally friendly production processes while at the same time maintaining out high quality standards.

## Expert opinion and information

Please visit our website for an overview of all available literature/reports on the product: [www.schuelke.com](http://www.schuelke.com).

\* The surface coverage of one wipe depends on the surrounding conditions and the structure of the wiped surface.



Schülke & Mayr GmbH holds a Manufacturer's Authorisation according to sect 13 para 1 German Drug Law and Certificates of GMP Compliance for medicinal products.



**schülke Headquarters**  
Schülke & Mayr GmbH  
Robert-Koch-Str. 2  
22851 Norderstedt, Germany  
Phone +49 (0) 40 - 52100 - 0  
Fax +49 (0) 40 - 52100 - 318  
[www.schuelke.com](http://www.schuelke.com)  
[mail@schuelke.com](mailto:mail@schuelke.com)

**Schülke & Mayr (Asia) Pte Ltd**  
Sime Darby Enterprise Centre  
10 Jalan Kilang #04-04/05  
Singapore 159410  
Phone +65 6257 2388  
Fax +65 6257 9388  
[www.schuelke.com.sg](http://www.schuelke.com.sg)  
[mail.sg@schuelke.com](mailto:mail.sg@schuelke.com)

**Schülke & Mayr (Asia) Sdn Bhd**  
Block A, Plaza Glomac  
Lot A-06-01& A-06-02  
No 6 Jalan SS7/19, Kelana Jaya  
47301 Petaling Jaya  
Selangor, Malaysia  
Phone +603 7885 8020  
Fax +603 7885 8021  
[www.schuelke.com](http://www.schuelke.com)