NL

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GB

The Bowie & Dick single use test pack is for the performance test in steam sterilizers. The Bowie & Dick single use test pack consists of pigments not containing any heavy metals. The indicator system is odorless and does not lose colour. To provide perfect performance of the highly sensitive indicator, these directions for use should be read and followed carefully.

1. Before starting the test run, warm up the sterilizer with an empty chamber run.
2. Put the Bowie & Dick single use test pack horizontally on a low area of the chamber of the sterilizer, e. g. above the drain of the empty chamber and nearby the door of the chamber.
3. Start the Bowie & Dick test run. After finishing the complete test run (134°C. for 3 ½ minutes *) the single use test pack should be removed immediately from the chamber. Open the test pack, take out the testsheet and examine and document the result.
4. To examine the results of the test run, put the autoclaved testsheet on a brightly lit surface. Please note: evaluation of the autoclaved testsheet with a light source underneath is not appropriate. Compare the indicator's colour shade at the edge of the testsheet with the indicator's colour shade in the middle of the testsheet. If the sterilizer manages penetration with steam effectively and if it is supplied with steam of correct quality (sufficient small amounts of inert gases), the indicator's colour changes completely and uniformly from bright blue to pink. If steam penetration is not sufficient or the sterilizer is supplied with steam of incorrect quality (too large amounts of inert gases), the indicator's colour changes not completely. The colour at the edge of the testsheet is different from the colour in the middle of the testsheet. In this case, check your sterilizer.
5. Result documentation with signature and countersigning is lawful and enough.

*) Temperature and time are considered to match typical tolerances, see e. g. EN 285 and EN 13060. Applicable time for the test may be different according to national or validation requirements.